

SOCKIT![®]

Oral Hydrogel Wound Dressing

Patient Instruction Sheet

- Apply SockIt! when you begin to feel pain, or when pain begins to intensify. SockIt! should be used at least 4-6 times each day for a minimum of 3 days. In addition to providing pain relief, SockIt! is a wound dressing that prevents contamination and promotes optimal wound healing.
- Unscrew cap from syringe. Screw on curved tip for ease of application and to prevent waste. Use enough gel to just cover the affected area. Keep tongue away from treated area.
- Remove curved tip and rinse. Replace screw cap on syringe. Retain remaining SockIt! and curved tip for subsequent applications. Store at room temperature.
- Use SockIt! first before resorting to prescription narcotics as the need for prescription medication is usually significantly reduced with use of SockIt!.
- SockIt! may be used as often as necessary. It is gentle to injured tissues. It contains no drugs, is non-toxic, and is safe to swallow. There are no medical, pharmacological or age restrictions associated with SockIt!.
- A very thin layer of SockIt! will adhere to the tissue and provide pain relief. Some of the gel will wash away and be swallowed. You will know when all the gel is gone by the return of or increase in discomfort.
- If you have any questions regarding the use of SockIt!, please call 800.755.0044.